mobilux[®] DIGITAL Touch HD

 Windows Vista™

 Windows[®] 7

 Windows[®] 8

Anleitung Software Installation
Software Installation Manual
Notice d'installation du logiciel
Istruzioni per l'installazione del software
Instrucciones de instalación del software
Handleiding software-Installatie
Vejledning software-installation
Installationsanvisning för programvara
Anvisning programvareinstallasjon
Ohjelmiston asennusohje
Instrukcja instalacji oprogramowania
Návod k instalaci software
ソフトウェア インストールの手順

CE

 ESCHENBACH

Deutsch.....	3
English.....	7
Français.....	11
Italiano.....	15
Español.....	19
Nederlands	23
Dansk	27
Svenska.....	31
Norsk.....	35
Suomi.....	39
Polski.....	43
Česky	47
日本語.....	51

Deutsch

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English


The "mobilux Digital" software supplied enables the live image from the camera to be displayed on a computer screen via the USB connection. Here we explain how to install mobilux Digital under Windows Vista.

Installation

1. Ensure that the SD card supplied is inserted in the reading device card slot. Switch the reading device on.
2. Connect the reading device to a USB connection on a suitable computer using the USB cable supplied.
3. Touch the SD card symbol:


4. Open a Windows Explorer window (this opens automatically using Win8), and locate the new drive called "ESCHENBACH". Open this drive and execute the "mobilux Digital.msi" file to start installation.
5. The installation wizard window opens: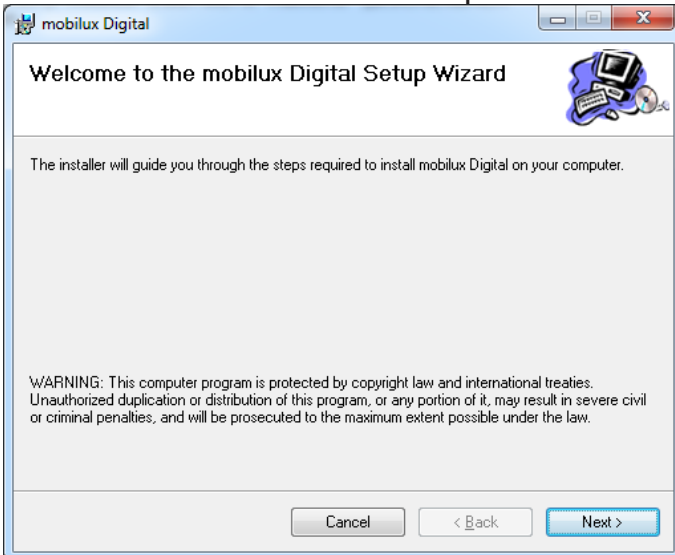

Click "Next" to continue the installation.
Click "Cancel" if you want to quit the installation.

6. Selection of installation folder: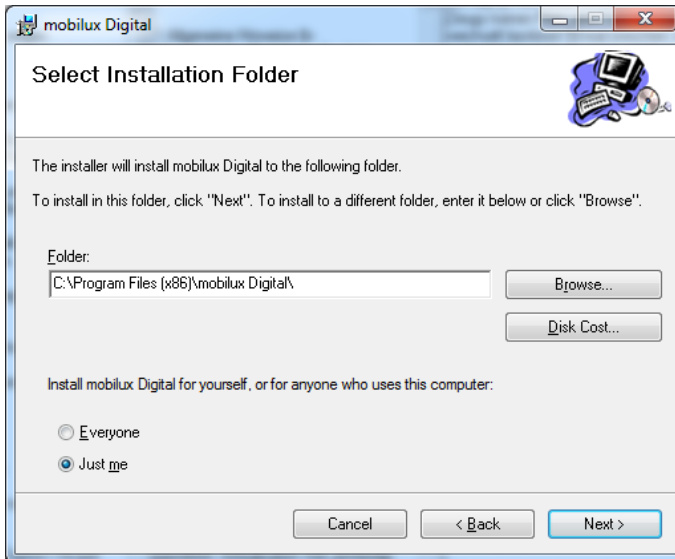

The folder for the file to be installed is preset. Click "Browse..." if you want to specify a folder other than the preset folder as the installation location. If you click "Disc Cost ...", you will see an overview of the available drives and their memory capacity.


Apart from this, you can specify whether the program is to be installed just for yourself (Just me) as a user or for every user (Everyone) on the computer in this window. To change this setting, click the grey point left of the required option.

Click "Cancel" if you want to quit the installation.

Click "Back" to return to the previous installation step.

Click "Next" to continue the installation.


7. Confirming the installation: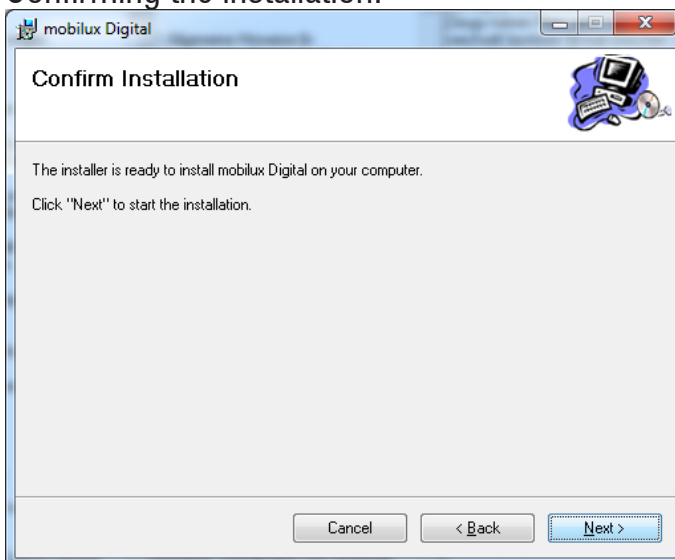

Click "Next" to carry out the installation immediately.

You will be shown a message about User Account Control depending on your computer configuration. Confirm this by clicking the "Allow" or "Yes" fields (Win7 & 8). The installation will be carried out.

8. Concluding the installation: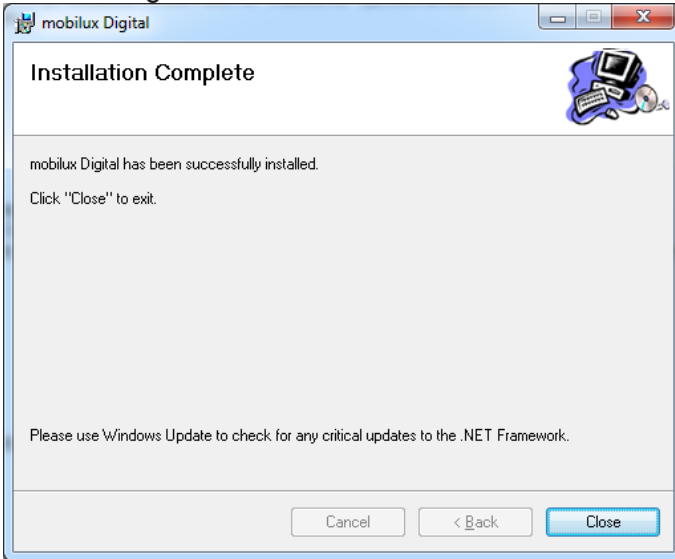

Once you see this window, the installation has been successfully completed. Click "Close" to close the window and complete the process.

Providing communication

1. Now tap the small SD card symbol bottom left on the reading device screen in order to access the USB function selection:


Tap the middle symbol to select the USB mode for communication between the reading device and the mobilux Digital software.

2. Start the mobilux Digital program. A window which shows the reading device live image will open on the computer screen. Place the reading device on the material to be read using its support surfaces and move it in the direction of reading during reading.

Please note that the reading device does not switch off after 3 minutes as stated in the instruction manual when operating the computer, since the power necessary for this operation is provided by the computer and does not affect the charging status of the built-in rechargeable batteries.

If you start the program without having selected the correct USB mode, a window with the following error message will be displayed: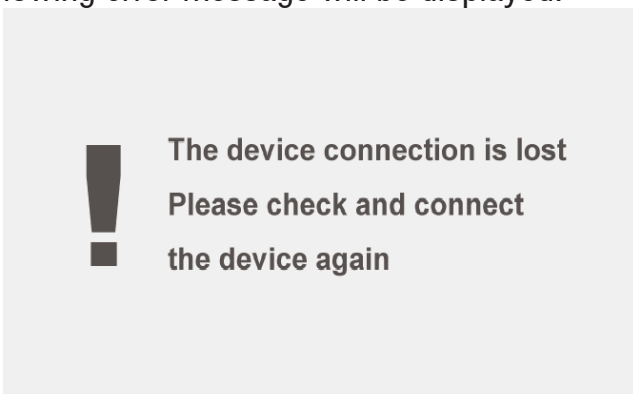

Now select the correct USB mode on the reading device by tapping this symbol:

3. To quit the program click the × top right in the mobilux Digital program window.
4. If you interrupt the USB connection between the reading device and the computer, the reading device will switch off automatically.

Additional information on mobilux DIGITAL Touch HD USB functionality

If you connect the reading device to a computer USB connection and switch it on, you will see the following 3 symbols on the reading device screen:

Tap on the  icon to set up the contents of the SD card in the reader as a new removable drive. Then proceed with the required file operation.

Tap the  symbol to enable communication between the reading device and the mobilux Digital software.

Tap the  symbol in order to work with the reading device while the internal battery is being charged using the USB cable..

To change the USB mode, tap the screen briefly to display the symbols if necessary. Then tap bottom left on the symbol for the current USB mode in order to re-display the selection of 3 symbols as described above. Then tap the symbol for the required USB mode.

USB® is a registered trademark of USB Implementers Forum, Inc.

Microsoft®, Windows® and Windows Vista™ are registered trademarks of or trademarks belonging to the Microsoft Corporation in the United States and/or other countries.

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